

The Only Thing You Ever Need



Bluetooth Free App available

Blood Glucose Monitoring System

GlucoNavii[®] PRO

Blood Glucose Monitoring System GlucoNavii® PRO



No Coding



**99.9%
Pure Gold Strip**



Easy Light™



**Strip Ejection
Function**



LCD Back Light*



REF 01GM60-1



REF 01GM61-1



Bluetooth Free App available

*Only GlucoNavii® Pro Bluetooth Blood Glucose Meter available.

Specifications

Result Range	10 - 600 mg/dL (0.6 - 33.3 mmol/L)
Sample	Fresh whole blood (Capillary, Venous, Arterial and Neonatal)
Sample Size	0.5 µl
Test Time	5 seconds
Assay Method	Glucose Dehydrogenase Biosensor
Memory	1,000 blood glucose tests
Battery Life	Around 1,000 tests
Operation Temperature	8°C - 45°C (46°F - 113°F)
Hematocrit	0 - 70%
Operation Altitude	Up to 11,549 feet (3,520 m)
Test Strip Storage Temperature	2°C - 32°C (36°F - 90°F)
Meter Storage & Transport Condition	-20°C - 50°C (-4°F - 122°F) and 10% - 93% RH
Connectivity	USB Connection, Bluetooth Connection (Optional)

GlucoNavii® PRO

Easy Light™

Target Glucose Range Set-up




	Arrow	LED
Low Limit 60~100mg/dL (3.3~5.5mmol/L)	↓	↔
High Limit 130~300mg/dL (7.2~16.6mmol/L)	↑	↔
Within Range	↕	—

➤ Gold Electrode & GDH-FAD Enzyme

Strip Advantages

“99.9% gold electrode”

Gold is the best stable material for electrical resistance, so it helps to get the best accuracy rather than other material like carbon.

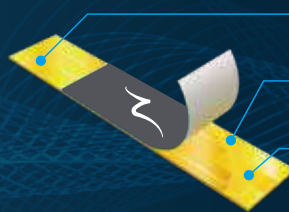

Wide Gold Electrode

Makes best accurate results

Laser patterning

Ensures an excellent precision

Fast Draw Technology

Makes test fast and easy

GDH-FAD enzyme glucose test strip

GlucoNavii® PRO GDH enzyme




- Can be tested by various blood samples (Capillary, Arterial, Venous, Neonate)
- Using specific enzyme, minimizing interference of disaccharide, polysaccharide, maltose, galactose, Icodextrin and more
- Minimize effect from oxygen saturation in blood
- More accurate result by providing wide HCT range (0%~70%)

➤ Bluetooth & GlucoNavii® App*

Platform For Digital Diabetes Management





- You can share your test results by using 'GlucoNavii® Application' with Healthcare Professional and Family.
- You can manage your records in the graph.

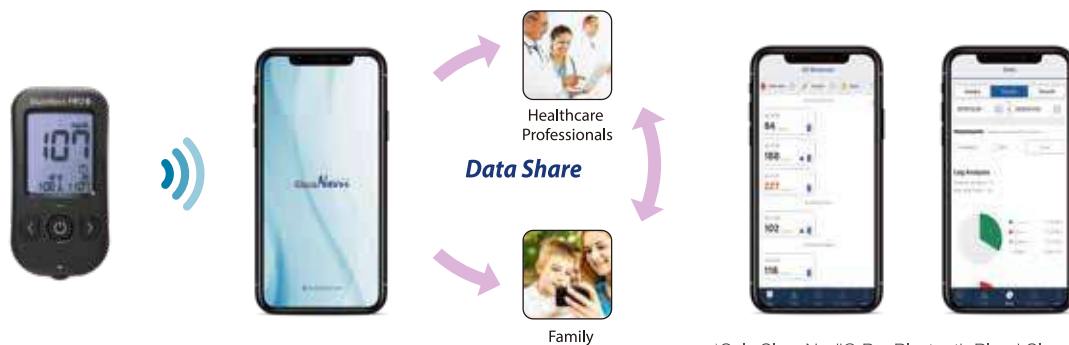

*Only GlucoNavii® Pro Bluetooth Blood Glucose Meter available.

➤ Accuracy & Precision

GlucoNavii® Pro exceeds ISO 15197:2013 requirement of 95% within $\pm 12\text{mg/dL}$ and $\pm 12\%$ of laboratory results respectively.

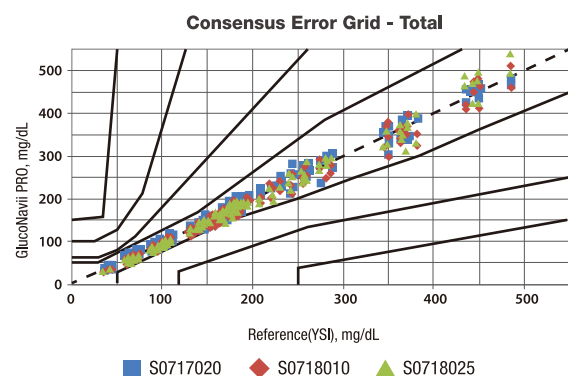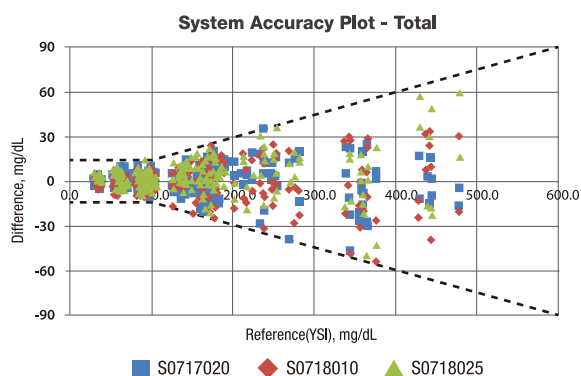

Blood Glucose Monitoring System
GlucoNavii[®] PRO



No Coding



99.9%
Pure Gold Strip



Easy Light[™]



Strip Ejection
Function

www.sdbiosensor.com



บริษัท ลาโบตรอน จำกัด 56/38-39 ซอย รามอินทรา 34
แยก 21 แขวงท่าแร้ง เขตบางเขน กรุงเทพมหานคร 10230

Labotron call center 
02-3624567